



To ensure that your breast form maintains its soft, supple feel and appearance, please give your form the same special care you give your body.

To protect your form:

- Store in its specially designed cradle
- Avoid sharp objects and direct sunlight
- Before handling your form, remove your rings and bracelets
- Beware of small punctures caused by scissors, pins, jewelry and pet's teeth and claws

Clean daily by gently washing your form using warm water and mild soap. Rinse thoroughly and pat dry with a clean cloth. Never wring, rub, use abrasive cleaners, acetone, alcohol or any other solvent on your breast form. Powders will tend to dry form, while perfumes may damage the protective outer layer. Do not hold/pick up form with your fingers. Hold the form with the palm/cup of your hand to prolong the wear of your breast form.